air n GO CLASSIC

SATELEC S.A.S. • A company of ACTEON Group
17, av Gustave Eiffel • BP 30216 • 33708 MERIGNAC cedex
FRANCE • Tel + 33(0) 556 34 06 07 • Fax + 33(0) 556 34 92 92 •
E.mail : satelec@acteongroup.com • www.acteongroup.com

J10261 • V3 • (10) • 10/2015 • NC10ZH020C

4 - COMPOSITION

AIRnGO «CLASSIC» powders: Sodium bicarbonate, hydrophobic silica, sodium saccharinate, excipients, natural essential oils or flavorings.

5 - PRECAUTIONS FOR USE

Before using the powder, we suggest you establish the general state of health of your patient.

- Adhere to the conditions of use recommended by the air polisher manufacturer.
- The patient and practitioner should wear appropriate protective accessories (goggles, masks).
- Do not aim the jet directly onto the gingiva.
- The patient should rinse the mouth using an antiseptic solution before and after treatment.
- Apply Vaseline® to the lips before polishing.
- Use a large aspirator cannula and hold it near the area to be treated (to reduce contamination by airborne bacteria).
- In the case of accidental contact with the skin or eyes, rinse thoroughly with water.
- Do not swallow.

6 - POSSIBLE SIDE EFFECTS

- Known allergy to one of the components.
- Patients suffering from endocarditis.
- Patients presenting an immune deficiency.
- Patients on antibiotics, undergoing chemotherapy, radiotherapy, etc.

- Patients on a low sodium diet.
- Diabetics, asthmatics, or patients suffering from respiratory problems.
- Hemophiliacs.
- Pregnant women or breastfeeding mothers.

7 - INSTRUCTIONS FOR USE

Follow the instructions for use delivered with your air polisher. In general:

- Maintain a «jet-surface» angle of between 45° and 60°.
- Maintain a «spray-surface» distance of about 6 mm.
- Make small circular movements on the tooth.
- Direct the jet towards the occlusal rim.

8 - STORAGE

Do not exceed the use-by-date shown on the external packaging. Close the bottle tightly after use (use contents quickly after opening). Discard the single-use dose sachet after use. Store at room temperature (< 25°C) away from damp and light.

9 - DISPOSAL OF UNUSED POWDER

Do not discard in drains or rivers. Do not discard with household waste. Caution: Federal Law in the United States restricts the use of this equipment solely to qualified dental health professionals, or those under their control.

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1 - PRESENTATION

AIRnGO «CLASSIC» polishing powders have been designed exclusively for supra-gingival prophylactic treatment. These powders can be used with all SATELEC® brand air polishers. AIRnGO «CLASSIC» powders: Natural flavors available as an option: peppermint, lemon, raspberry, cola or neutral.

Packaging of AIRnGO powders:

- 20 g unit dose sachets for single use only – Boxes of 50 sachets.
- 250 g multi-dose bottle with spout.

2 - INDICATIONS

- Removal of dental plaque, (orthodontic treatment or before whitening), stain removal.
- Removal of temporary cement residues.
- Polishing of teeth after scaling.
- Inter-proximal cleaning, cleaning of fissures and grooves.
- Cleaning of bands and brackets in the mouth.
- Cleaning and preparation of the dental surface before etching.

3 - PROPERTIES

The abrasive properties of AIRnGO powders do not damage either the enamel or the gingiva. AIRnGO «CLASSIC»: Painless and without causing excessive bleeding, it is appropriate for supra-gingival maintenance treatments. Available in various flavors, it gives a sensation of freshness in the mouth.