Suprasson® & Suprasson® Lux

User's manual / Manuel d'utilisation / Manual de uso
Bedienungsanleitung / Istruzioni d'uso / Handleiding voor de gebruiker

SATELEC

English	KIT INSTRUCTIONS	4
Français	NOTICE COFFRET	18
Español	MANUAL DE USO	28
Deutsch	BEDIENUNGSHINWEISE	40
Italiano	ISTRUZIONI D'USO	52
Neederlands	INSTRUCTIES BIJ DE KIT	64

ENGLISH

I - Installation 5

II - Maintenance 7

III - Liability 16

I - INSTALLATION

1. 1 Suprasson handpiece connection

a) Suprasson handpiece (Fig. 1 & 2)

Connect the handpiece by plugging it into the cable socket.
Do not rotate.

b) Suprasson Lux handpiece (Fig. 3)

The handpiece is just plugged into the cord socket by positioning the cord mark opposite the handpiece mark.
Do not rotate.

1. 2 Tips mounting

The Suprasson handpiece is a high technology instrument requiring precautions for its use. The tips must be screwed and moderately tightened by means of the corresponding wrench (Fig. 4).

CAUTION: Correct tip vibration requires perfect tightening without overtorquing beyond the thrust point. Remove the tip after each procedure to avoid self-jamming.

1. 3 Sterijoint® connection

- Install the Sterijoint® connector (Fig. 5) or Sterijoint® Lux (Fig. 6) onto the handpiece and connect the cord (Fig. 7-1).
- Connect the irrigation line to the small tube on the Sterijoint® connector (Fig. 7-2).

II - MAINTENANCE

2. 1 Suprasson and Suprasson Lux handpiece maintenance (Fig. 8-9)

a) Cleaning

Handpiece bodies must be cleaned and disinfected (e.g. with alcohol : 96% V/V ethanol or dental disinfecting towelettes).

The threaded cavity can be cleaned by means of a cotton swab soaked in alcohol. The handpiece must not be cleaned in an ultrasonic bath.

NB: according to version:

Suprasson handpiece (Fig.1): the handpiece must not be cleaned in an ultrasonic bath.

Suprasson handpiece with disconnectable nozzle (Fig. 2): front part of the handpiece plastic casing can unscrew to allow better access to the thread

for more efficient and easy cleaning with cotton wool soaked in alcohol.

Caution :

- Do not disconnect the handpiece while the instrument is powered and the control pedal is simultaneously activated.
- Removing the Suprasson Lux handpiece, when pulling off the handpiece, always hold

- the control ring of the supply hose, not the
hose sleeve.
- b) Cleaning of the fiber optic surfaces
For Suprasson Lux handpiece only
In order not to scratch surfaces, blow away dirt
particles with spray/air jet.
Before sterilization, clean the surfaces with a
cotton swab or soft cloth and alcohol.
- c) Sterilizing conditions
Suprasson handpiece
Sterilizing conditions according to equipment:
autoclave.
It is strongly recommended to sterilize the
handpiece, with the nozzle removed, in a class B
or N autoclave.

Suprasson Lux handpiece

The Suprasson Lux handpiece is not
chemiclavable.

Sterilizing conditions according to equipment:
autoclave

- Separate the handpiece from the cord and remove the tip.
- Put the handpiece in a sterilizable cloth or bag, if needed.
- Take the handpiece out of the autoclave as soon as sterilization is complete.
- Carefully dry the electrical contacts of the handpiece and of the cord connector before use.

d) Sterilization

Sterilization: wet heat:
10 134°C at 2 bars (200 KPa) for 18 mins.

Warning: The water circuit of the Suprasson handpiece and tip assembly must be flushed with water for 20 to 30 seconds after each use.

Caution:

- Prior to sterilization, use compressed air to remove any irrigation fluid left in the Suprasson handpiece.
- Always sterilize the Suprasson handpiece

without the tip mounted.

- In use and during successive sterilization procedures, check the Suprasson handpiece for any sign of external damage before you next use it.

Suprasson handpiece

The handpiece holds an irrigation circuit gasket ring in its rear part (1.15 x1) accessible and visible thanks to a cut-up of the rear bush. As a

result, the gasket ring can be easily replaced if it is damaged or excessively worn. Have it replaced or replace it yourself (ref.: F12304: 3 water circuit gasket rings + ring fitting tool). With disconnection/re-connections and/or sterilisations, this gasket ring should be lubricated with a grease for dental instrumentation in order to prolong its efficacy and prevent internal leaks.

12

Suprasson Lux handpiece

The Suprasson Lux handpiece color may change over time without the performance characteristics and specificities being impaired in any way.

2. 2 Trouble shooting

	Fault	Possible cause	Remedy
No operation		Presence of fluid or moisture between handpiece, cord and/or Sterijoint®	Dry any moisture, especially on electrical contact
No spray		Incorrect water tube connection No water pressure	Check water supply Check pressure

No water at tip but vibration present	Obstructed tip Wrong tip selection Incorrect spray adjustment	Remove obstruction Check tip Adjust spray
Insufficient power Low vibration	Worn or deformed tip Incorrect use: wrong angle of application or inadequate pressure on tooth Presence of fluid or moisture between handpiece, cord and/or Sterijoint	Replace tip See tip utilization method Wipe electrical contacts dry

No ultrasonic output	Incorrect tip tightening Defective connector contact Cut wire in handpiece cord	Tighten with wrench Clean cord contact Return to customer service for cord replacement
Water leak between handpiece and cord or at front/back junction of handpiece	Worn sealing gasket (1.15x1) in handpiece	Replace seal

III - LIABILITY

The manufacturer's liability shall not be pledged in the following cases:

- Maintenance or repair by persons not authorized by the manufacturer.
- Use on an electrical installation that does not comply with the regulations in force.
- Other uses as those specified in this Manual.
- The manufacturer reserves the right to modify the device and/or the instruction manual without prior notice.

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